-  Water Distiller (GB)
-  Distillateur d'Eau (FR)
-  Wasserddestillator (DE)
-  Distillatore d'Acqua (IT)
-  Destillador de Agua (ES)
-  Vatten Destillator (SE)
-  Veden Tislauslaite (FI)
-  Destillationsapparat (DK)
-  Water Distillator (NL)
-  Desztillátor (HU)
-  Destilátor (CZ)

DIST

CE MARK

GB	Applicable European directives and standards	
FR	Directives et normes européennes applicables	
DE	Anwendbare europäische Richtlinien und Normen	- Directives: Low Voltage Directive 2014/35/EU EMC Directive 2014/30/EU
IT	Direttive e norme europee applicabili	
ES	Directivas y estándares europeos aplicables	
SE	Tillämpliga europeiska direktiv och standarder	
FI	Sovellettavat eurooppalaiset direktiivit ja standardit	- Standards: EN 60335-1:2012+A11+A13+A1+A14+A2+A15 EN 60335-2-15:2016+A11+A1+A2+A12 EN 62233:2008
DK	Gældende europæiske direktiver og standarder	EN IEC 55014-1:2021
NL	Toepasselijke Europese richtlijnen en normen	EN IEC 55014-2:2021
HU	Az alkalmazandó európai irányelvek és szabványok	EN IEC 61000-3-2:2019+A1:2021 EN 61000-3-3:2013+A1:2019+A2:2021
CZ	Platné evropské směrnice a normy	

Megahome

Megahome Corporation

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
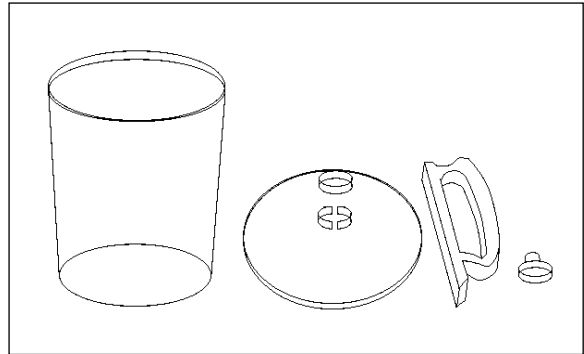
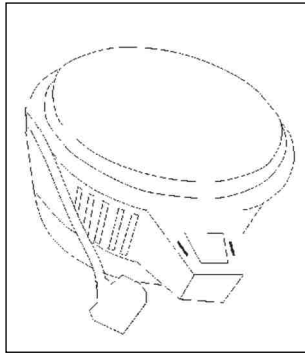
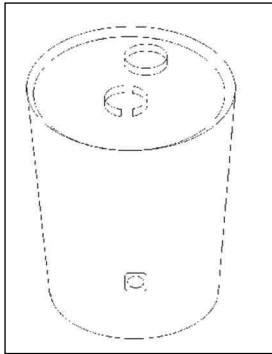
Contents of the package


Accessories

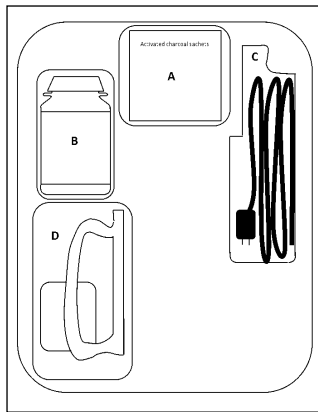
- A-6× activated charcoal sachets
- B-1× 250 gram cleaner
- C-1× power cord
- D-1× nozzle, handle and lid for bottle


Assembly of the bottle

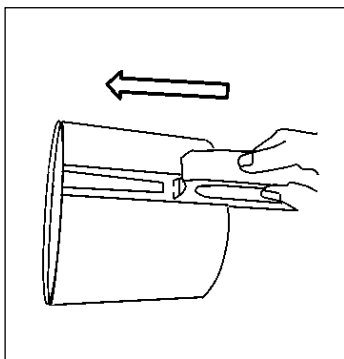1

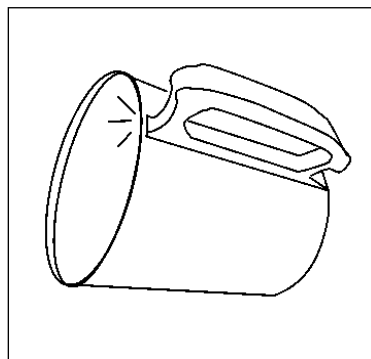2

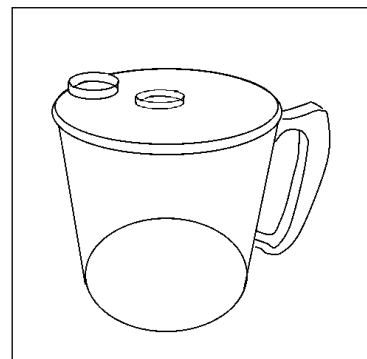
3

DIST water distiller makes the purest and cleanest water in the most natural and effective way.

HOW IT WORKS

When the distiller is turned on, the water temperature rises gradually ($\geq 100\text{ }^{\circ}\text{C}$). Chemicals, heavy metals and other contaminants, will be left as a residue at the bottom of the stainless steel tank. Impurities and unpleasant odours will be removed as steam collects on the surface of the cooling duct, which forms distilled water droplets and passes through the activated charcoal sachet. You then have sweet, pure water!

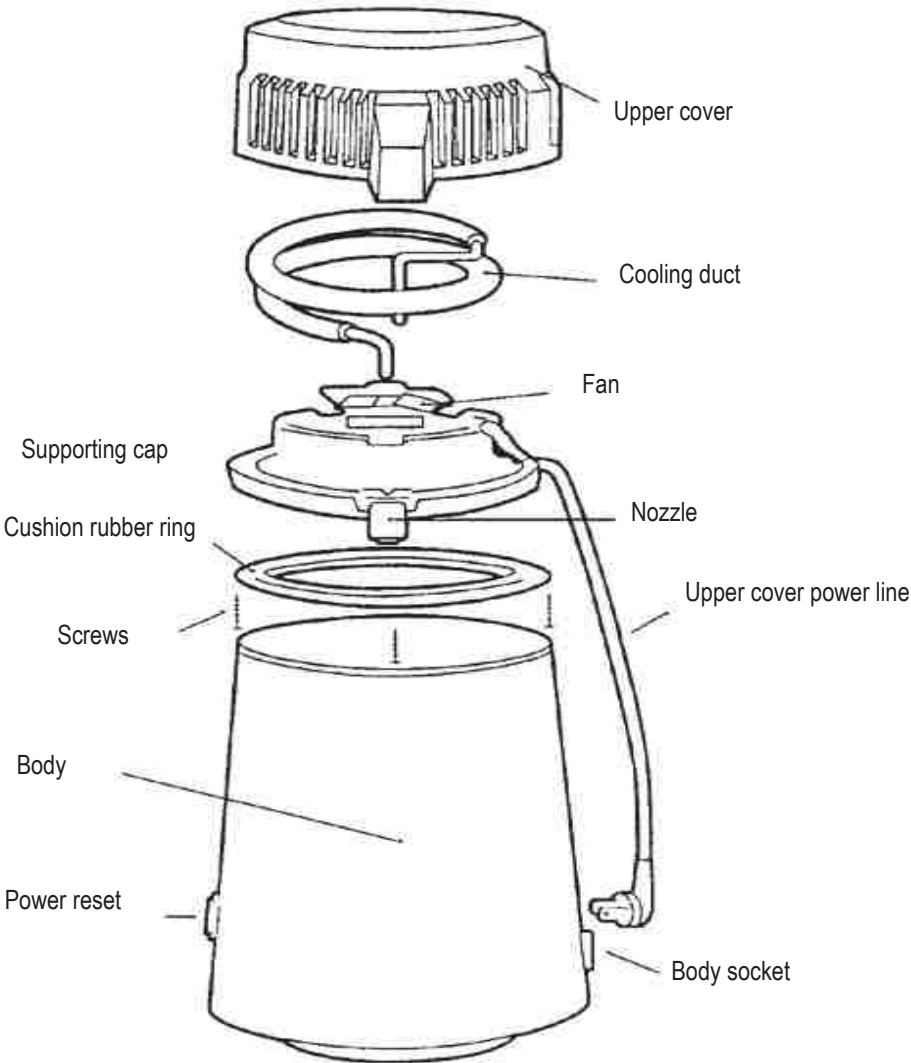Please don't drink the first 2000cc.of water as this may contain any residues after manufacture.


INSTRUCTION

Activated charcoal installation

(not required if the water is used in a sterilizer):


1. Remove the water dispenser from the safety cover by pressing the two wings.
2. Fold the activated charcoal sachet in half and put it into the water dispenser.
3. Reinstall the water dispenser onto the water outlet of the safety cover.

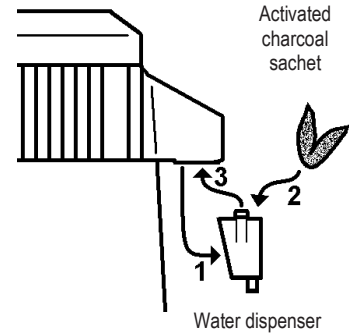Each activated charcoal sachet can be used for up to one month.


Pour water into the stainless steel tank

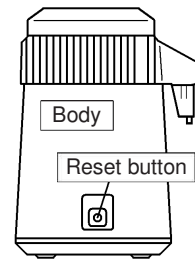Use hot/cold water to fill the tank and make sure that you do not go beyond the full line. If you use hot water, you can shorten the time needed to distil water.


Starting up

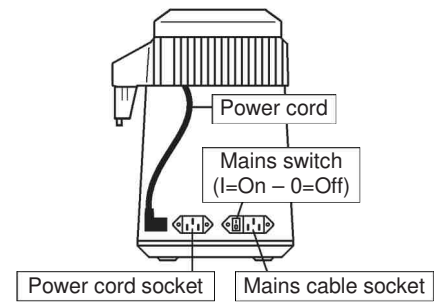Plug the mains cable into the socket. Secure the upper cover and plug the power cord into the socket. Turn the mains switch ON. Place the collection bottle and the protective cover in a secure place and make sure they are levelled. Press the reset button (when first used the button is automatically set in the "on" position, so there is no need to switch it on for the very first use).

The process starts when you press the reset button and it will stop automatically at the end of the process. If you wish to terminate the process, unplug the power cord.

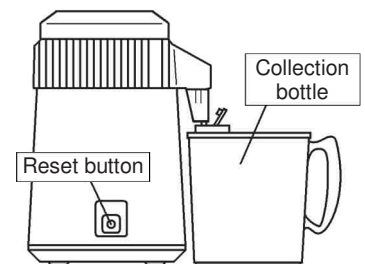The power supply is turned off automatically when the distillation process is completed.

Approximately an hour after you have turned on the power switch, the distilled water begins to drip down into the collection bottle. It takes 5-6 hours to distil four litre of water.

When the power supply is cut off, you may move onto the next tank immediately.

Distilled water

For proper storage place the distilled water in the refrigerator in a sealed container. Before use shake the bottle to aerate the water.

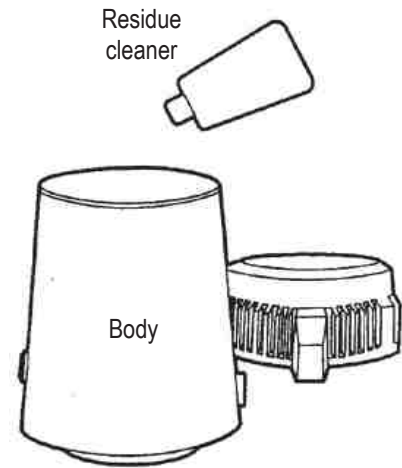
MAINTENANCE

Use household neutral detergent to clean the surface of the body and keep it dry (the rubber cushion within the upper cover is set with a specific direction; please do not remove it on your own).

To clean the body

The interior of the stainless steel tank is smooth. Never use abrasive cleaners on your distiller. After each cycle rinse the inside of the boiling chamber with clean water. To remove heavy deposits, use the distiller cleaner as follows:

1. Pour hot water into the stainless steel chamber until it covers the deposits.
2. Pour one to four spoonfuls of cleaner (based on the amount of deposits) into the tank.
3. Leave the upper cover off, plug in the main power plug, and turn on the power reset to heat the stainless steel tank until boiling.
4. Discard the water fifteen to twenty minutes later, then wash the interior with clean water.
5. Repeat the process until the chamber is clean.
6. Small amounts of residue are harmless, and you can continue to use it safely.

Cooling duct

If the condensing coil is clogged with dust, the efficiency of distillation may be affected. To clean the condensing coil, remove all three screws from the underside of the cover.

Use a tooth brush or a vacuum cleaner to remove the accumulated dust.

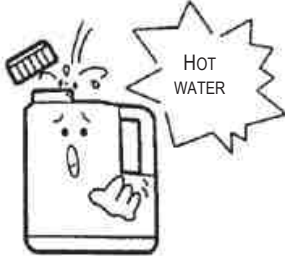
Others

“**Take care**” do not immerse the motor in the water when you are cleaning the cover. We recommend you regularly add a little oil on the motor shaft to help maintain the motor in good working order.

...FOR YOUR SAFETY


GB

Place the distiller on a dry, flat and heat-tolerant surface and keep it away from any heat source. Because the cooling fan uses air to cool it, we suggest you place it in a well-ventilated space.


Do not use a plastic bottle to store hot water

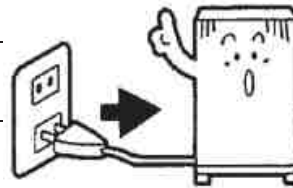Unplug the power line if you wish to terminate the distillation process prematurely.



Do not remove the upper cover when operating the distiller.

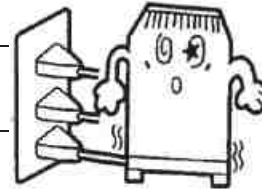Use a power point with proper voltage and do not share the same outlet with other electrical appliances to prevent overloading.

Do not let the children operate the distiller to avoid accidents.

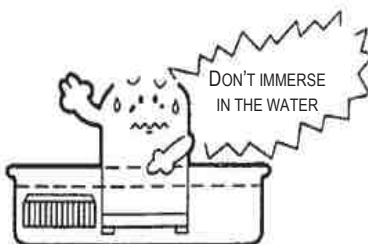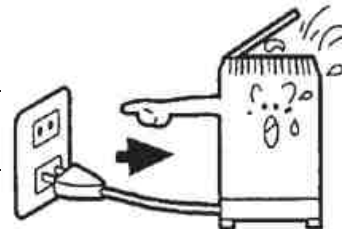Unplug the main power cord when filling or cleaning the distiller or when not in use.

Never immerse any part of the distiller in the water for the purpose of cleaning.

CONDITIONS OF GUARANTEE

This product is made of high quality material and great care has been taken in its manufacture. It is designed to give good service, provided it is properly operated and maintained. For your safety, please follow the instruction manual to prevent any possible damage.

The warranty shall not apply in the following circumstances: if it has been dismantled by an unauthorised person, connected to an incorrect mains voltage, or not used according to operating instructions, or to accidental damage.

TROUBLESHOOTING

Conditions & Points to check:

Not making any distilled water

- Main power cord not plugged into power point or lid cord not plugged into base of the distiller.
- No power to electrical outlet.
- Reset button not pressed.

Not making enough distilled water during each cycle

- Water not filled to maximum fill line.
- Lid gasket not sealing properly or up the wrong way.
- Collection bottle not placed directly under the spout.

Water or steam leaks from spout




- Carbon sachets not properly placed in the nozzle.
- Collection bottle not correctly lined up with distiller.

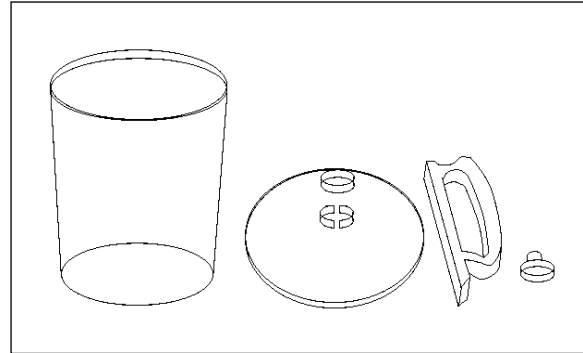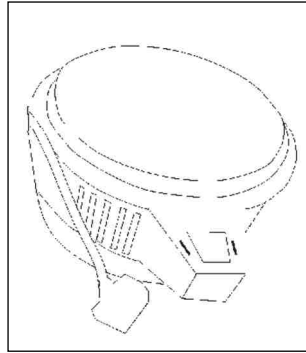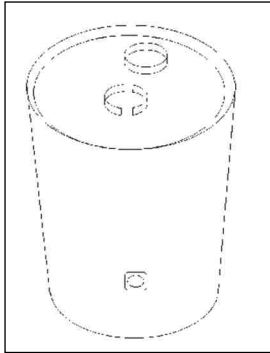Or contact your distributor for further advice.

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