





Quick Start Manual

Set up eLog

ioDent®

Log in to the ioDent® account


- 1 Open "Reprocessing" page

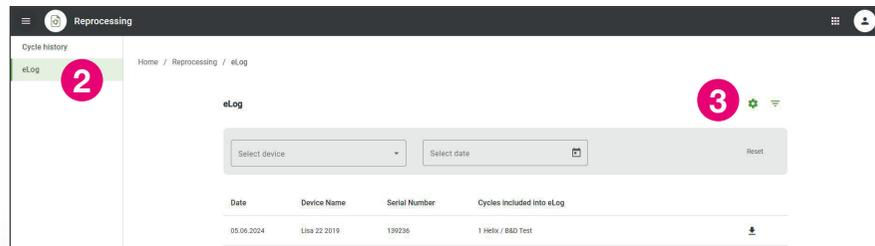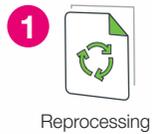


- 2 Click on eLog option

First access - important steps:

- 3 Click on the gear icon
- 4 Choose your preferred settings:
 - > Decide which cycles to display in eLog
 - > Select the time period
 - > Click on "Save" button
- 5 Activate eLog with status toggle-button:
 - > Click on the grey bar to turn the colour to green
 - > Click on "Save" button

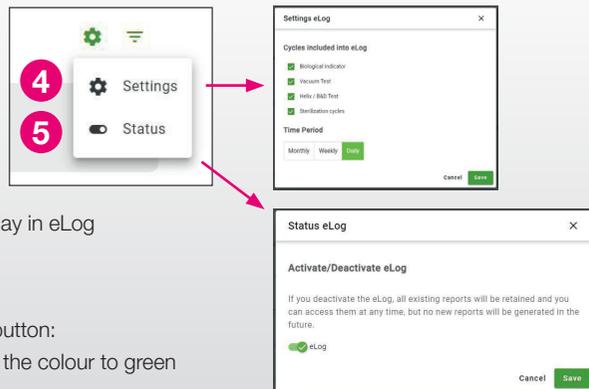

W&H Steri App

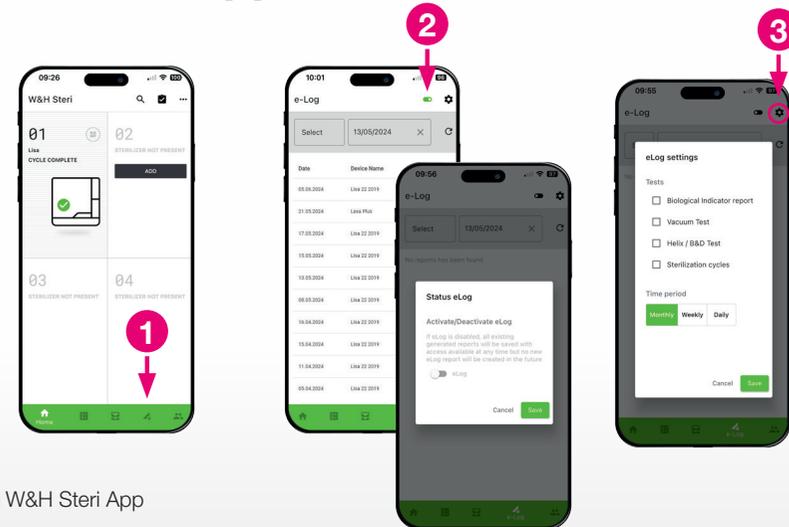


Open W&H Steri App

- 1 Tap fourth icon in the bottom bar
- 2 Activate eLog with status toggle-button:
 - > Tap the grey toggle-button to turn the colour to green
 - > Tap "Save" button
- 3 Tap gear icon and choose your preferred settings:
 - > Decide which cycles to display in eLog
 - > Select the time period
 - > Tap "Save" button

Become familiar with eLog

How to perform test traceability

Before cycle:

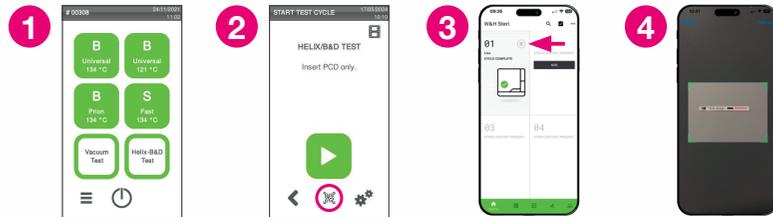

- 1 Select the cycle on the sterilizer menu
- 2 When you are on the sterilizer's start cycle screen, take your W&H test and scan the QR code on it with the following options.

Option 1: Linking the W&H test to the cycle using the W&H Steri App


- 3 Tap the QR code icon
- 4 Scan the QR code on the W&H test with the camera

Option 2: Linking the W&H test to the cycle using the barcode scanner

- 5 Scan the QR code on the W&H test with the barcode scanner connected to your sterilizer



After cycle is completed:

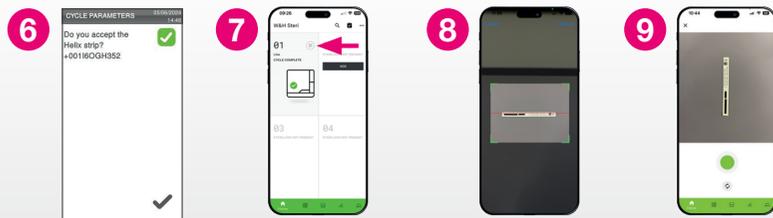

- 6 Visually inspect the test result and confirm on the sterilizer that the test is passed according to IFU

Linking the W&H test picture to the cycle using the W&H Steri App:


- 7 Tap the QR code icon
- 8 Scan the QR code on the W&H test with the camera
- 9 Take a picture of the test

How to display eLog

Based on your settings, you will find your eLog reports arranged daily, weekly or monthly

From ioDent®:

- 1 Access eLog from ioDent®
- 2 Download eLog report by clicking on the download icon
- 3 Downloaded eLog PDF document

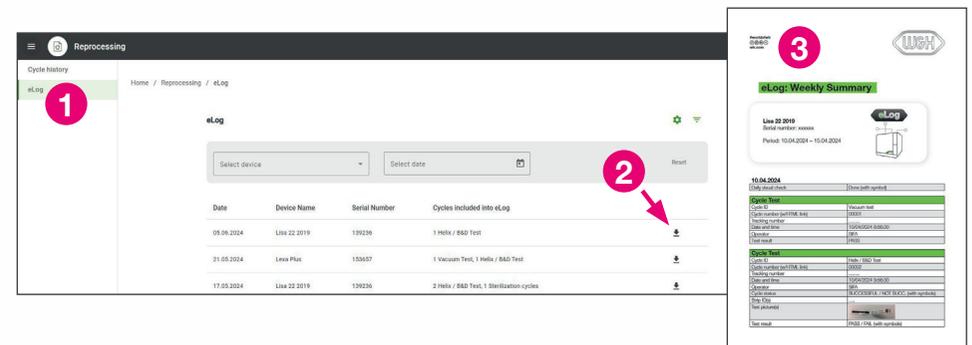

From W&H Steri App:

- 4 Tap fourth icon in the bottom bar
- 5 Tap eLog report to be downloaded
- 6 Your smart device will give you the option to send, share or locally save your eLog report

