## EN (English)

**INTENDED PURPOSE:** Bepro Disinfectant R is a ready-to-use solution for the disinfection and cleaning of rotary dental and surgical instruments made of stainless steel, tungsten carbide, diamond grit, ceramic, polymer and silicone. Bepro Disinfectant R has a very broad effective spectrum and is fast acting. Bepro Disinfectant R is easily biodegradable.

**EFFECTIVE SPECTRUM:** Bactericidal (EN 13727:2012+A2:2015\*, EN 14561:2006\*) 30 sec; tuberculocidal (EN 14348:2005\*, EN 14563:2008\*) 60 sec; mycobactericidal (EN 14348:2005\*, EN 14563:2008\*) 60 sec; yeasticidal (EN 13624:2013\*, EN 14562:2006\*) 30 sec; fungicidal (EN 13624:2013\*, EN 14562:2006\*) 60 sec; enveloped viruses (EN 14476:2013+A1:2015\*, EN 17111:2018#) 15 sec; virucidal (EN 14476:2013+A1:2015\*, EN 17111:2018#) 30 sec

<sup>^</sup> Tested under low organic load / <sup>\*</sup> Tested under high organic load / <sup>#</sup> Tested under low and high organic load
**INSTRUCTIONS FOR USE:** For your protection, wear a laboratory gown, household gloves and safety goggles. Immerse the rotary instruments immediately after use in the disinfectant solution. Adhere to but do not exceed indicated contact time. After disinfection, leave the rotary instruments to dry. Do not rinse with water. Visually check instruments after the disinfection for contaminants. If necessary, remove contaminants with a brush and repeat the disinfection process. Replace working solution daily or when visibly contaminated. The disinfectant solution is also suitable for use in an ultrasonic cleaning device. Adhere to reprocessing recommendations of the instrument manufacturer. Use only with alkaline and alcohol compatible instruments. Not compatible with aluminium. A list of compatible and incompatible materials is available on our website. If you are uncertain as to the material composition of the medical device, contact the manufacturer before use.

**LIMITATIONS:** FOR PROFESSIONAL USE ONLY. Not for final disinfection of critical medical devices. Sterilize medical devices after disinfection. Do not mix with other products. For fungicidal and virucidal efficacy, perform disinfection on previously cleaned instruments only.

**HANDLING:** WARNING H226 Flammable liquid and vapour. H315 Causes skin irritation. H319 Causes serious eye irritation. H336 May cause drowsiness or dizziness. P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P261 Avoid breathing fume/mist/vapours/spray. P280 Wear protective gloves/protective clothing/eye protection/face protection. P303 + P361 + P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower. P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P501 Dispose of contents/container in accordance with local and national regulations. For more information, please refer to Safety Data Sheet available on our website.

**STORAGE:** Store in cool, dry and well-ventilated facility. Keep out from direct sunlight, heat, and open flame. Storage temperature: < 30°C

**COMPOSITION:** 100 g contains 32 g isopropanol, 0,7 g 2-phenoxxyethanol, 0,2 g quaternary ammonium compounds, corrosion inhibitors, auxiliaries and water.

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