bepro suction CD

#whsterilization

wh.com

EN Liquid concentrate with enhanced disinfecting effect for the daily cleaning and maintenance of dental aspiration systems with and without amalgam separator.

- free of aldehydes, phenols and phosphates
- foam reduced
- 2 % application
- high cleaning power
- VAH/DGHM certified (instrument disinfection)

The active substances, harmonizing perfectly with each other, are particularly compatible with materials, have a deodorant effect and reduce the germ loading of the system.

EFFECTIVE SPECTRUM

- Bactericidal
- Yeastcidal
- Virus-inactivating against enveloped viruses (incl. HBV, HCV, HIV, Herpes simplex, Influenza incl. H1N1 + H5N1, BVDV, Vaccinia) as well as non enveloped Adeno viruses.

Application:

For the reliable cleaning, disinfection and maintenance it is recommended to use Bepro Suction CD for 3 consecutive working days 1 - 2 times each day (e. g. noon, evening). For the remaining 2 working days the application with is recommended.

For manual application with the Bepro Suction Mixing Cup:

2 L of a 2 % ready-to-use solution are required for one application. For this purpose dose 2 x 20 ml concentrate by using the dosage chamber of the dosage bottle and fill up in the Bepro Suction Mixing Cup with water to 2 L of a ready-to-use solution. For this it is recommended to use 40 - 50 °C warm water. Afterwards aspirate 1 L of the ready-to-use solution through the aspiration tubes and pour the remaining 1 L into the spittoon bowl. Thereafter let take effect for at least 60 minutes and rinse the aspiration tubes and the spittoon bowl thoroughly with water just before starting work.

Devices with integrated chemical tube cleaning:

Fill concentrate directly into the detergent tank of the device. Adjust mixing ratio to 2 %. For details, please refer to the manufacturer's instructions.

Clinical data:*

- Dr. H. Brill, Hamburg, VAH incl. EN 14561 + EN 14562, 2006-10
- Prof. Dr. R. Schubert, Frankfurt/M., VAH, 2007-02
- Dr. J. Höffler, Hamburg, EN 13624 + EN 13727, 2009-05
- Dr. J. Steinmann, Bremen, HBsAg-procedure, 2003-10; BVDV, 2003-11; Vaccinia, 2006-04; Adeno, 2006-05

*Expert opinion according to EN-standards under dirty conditions and at room temperature.

Composition

5-15 % cationic surfactants,
< 5 % amphoteric surfactants,
< 5 % non-ionic surfactants,
Contains:
2-Aminoethanol (141-43-5), Benzalkonium chloride (85409-22-9)

Ecological compatibility

According to OECD regulations the active substances in Bepro Suction CD are well biodegradable.

Special notes

Danger. Causes severe skin burns and eye damage. May cause respiratory irritation. Wear protective gloves/protective clothing/eye protection/face protection. IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water [or shower]. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/doctor. Bepro Suction CD must not be mixed with other detergents or disinfectants.

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