For the preparation of bepro suction ready-for-use solutions for the maintenance and cleaning of dental aspiration systems.

1. USER INSTRUCTIONS

Please attend to the following instructions to guarantee a trouble-free, economic and safe work.

2. SYMBOLS

	Serial number XXXXXX (6 digits)
	Please follow the instructions for use
	CE symbol for medical device class I
	Please note
	Order number
	Manufacturer
	Date of Production
	Distributor
	Machine readable code in HIBC format

3. ORDER DATA

REF 19500299 bepro Suction Mixing Cup with double adapter

REF 19500297 single adapter

4. TARGET GROUP

This instruction for use is intended for all users (doctors, dentists and professionals in charge of the processing of medical device as well as hygiene professionals) in medical and dental facilities.

5. USE THE BEPRO SUCTION MIXING CUP ONLY WITH THE BEPRO SUCTION SERIES

REF 19500253 bepro suction CM
for weekly acid contra-cleaning and maintenance


REF 19500254 bepro suction CD
with extended disinfection effect for daily cleaning and maintenance

6. APPLICATION OF BEPRO SUCTION MIXING CUP

2 L of a ready-to-use solution are required for one application.

For this purpose dose bepro suction CD (40ml) or bepro suction CM (100ml) concentrate (Fig. 1) and fill up in the bepro Suction Mixing Cup with drinking water (room temperature) to 2 L (Fig. 2).

Afterwards aspirate 1 L of the ready-to-use solution through the aspiration tubes (Fig. 3) and pour the remaining 1 L into the spittoon bowl (Fig. 4). Thereafter let take effect for at least 30 minutes and rinse the aspiration tubes and the spittoon bowl thoroughly with water just before starting work.

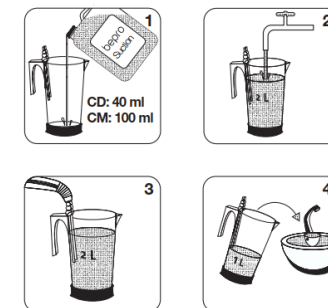Application only by qualified personnel with adequate protective clothing.
When using the bepro Suction Mixing Cup with bepro suction CM and bepro suction CD, please follow the instructions on the respective product label.

7. MAINTENANCE

Visible soiling can be removed with bepro suction CM or bepro suction CD and a plastic brush.

8. REPORT OF INCIDENT

Please report any serious incident that may occur in connection with the product to W&H Sterilization (office.sterilization@wh.com) and to the national authority.

9. DISPOSAL

The bepro Suction Mixing Cup must be disposed of with the residual waste, the packaging can be recycled.

10. NOTES

Product information (safety data sheets, instructions for use etc...) is always available at wh.com or by scanning the following code

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