

Instructions for use

Translation of the original operating manual



CE
perfecta
300 / 600

Contents

W&H Symbols	3 – 5
1. Introduction	6 – 7
2. Unpacking.....	8
3. Equipment supplied.....	9 – 11
4. Safety notes	12 – 13
5. Description / Starting operation / Operation – Foot control unit	14 – 17
6. Description / Assembly / Starting operation – Knee control unit.....	18 – 20
7. Description / Starting operation – Table control unit.....	21 – 22
8. Description of the motor handpiece / <i>E-Motor »E-type«</i>	23
9. General operation – Motor handpiece / Blow out function (Perfecta 600)	24 – 25
10. Description of the operating controls / Assemble support (optional)	26 – 27
11. Operating the operating controls – Reverse operation / Changing the speed / Bistable operation / Speed control operation (Perfecta 600).....	28 – 31
12. Removing and assembling the motor handpiece	32
13. Changing the rotary tools	33
14. Cleaning	34
15. Cleaning / changing the chuck.....	35 – 36
16. Error messages.....	37
17. W&H Accessories	38
18. Servicing.....	39
19. Technical Data.....	40 – 41
20. Recycling and Disposal	42
Letter of indemnity	43
CE-Declaration of conformity	44
Authorized W&H service partner	45

W&H Symbols



WARNING!
(Risk of injury)



ATTENTION!
(to prevent damage occurring)



General explanations,
without risk to
persons or objects











W&H Service

Only for USA

Caution: Federal law restricts this device to sale by or on the order of a dentist, physician or any other practitioner licensed by the law of the state in which he or she practices to use or order the use of the device.

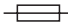
W&H Symbols

	Consult instructions for use		Do not dispose of with domestic waste	REF	Catalogue number
	Date of manufacture		Foot switch	SN	Serial number
	Electric fuse		Motor	V	Supply voltage of the unit
	Data Matrix Code, only for certain models, for product identification		Operating controls	AC	Alternating current
				VA	Electric power input of the unit
				A	Supply current
				Hz	Frequency of the alternating current

W&H Symbols



This way up

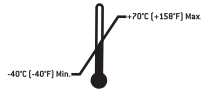


Temperature limit



Caution: Federal law restricts this device to sale by or on the order of a dentist, physician or any other practitioner licensed by the law of the state in which he or she practices to use or order the use of the device



Fragile, handle with care

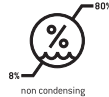

Humidity limitation



Keep away from rain



CE from manufacturer



Der Grüne Punkt
Identification mark of Duales
System Deutschland AG




Manufacturer



General symbol for
recovery/recyclable

1. Introduction

 **For your safety and the safety of your team**
These Instructions for use explain how to use your W&H product. However, we must also warn against possible hazardous situations. Your safety and the safety of your team are of paramount importance to us.

 It is therefore essential to read the safety notes on Pages 12 to 13.

Intended use

The control unit is intended for rotating or machining use with all material processed in a dental laboratory, whereby the handpiece is guided by hand.

 The control unit is **not** designed as a piece of **medical** equipment!
It is not permitted for use on patients!

Skilled application

The Perfecta is intended only for skilled application in the dental or medical field according to its purpose of use in compliance with the valid health and safety at work regulations, the valid accident prevention regulations as well as in compliance with these instructions for use. Non-compliance with these instructions or use of accessories and spare parts that are not approved by W&H invalidate all claims under warranty and any other claims.

Introduction

The Perfecta is in the condition as supplied by us



- > safety checked
- > carries the UL mark of quality
- > suppressed in accordance with pertinent standards.

This declaration is not valid for unintended external or internal attachments and the like.




Responsibility of the manufacturer

W&H Dentalwerk Bürmoos can only accept responsibility for the safety, reliability and performance of the Perfecta when there is compliance with the following directions:

-  > Consult Instructions for use
-  > *This Instructions for use also applies to the »E-type«-control units. »Motor handpiece« is to be understood accordingly as »E- Motor with ISO coupling« (Perfecta 300).*
- > The Perfecta has no components which can be repaired by the user. Assembly, modifications or repairs must only be undertaken by an authorized W&H service partner (see page 45).
- > Unauthorised opening of the equipment invalidates all claims under warranty and any other claims.


2. Unpacking

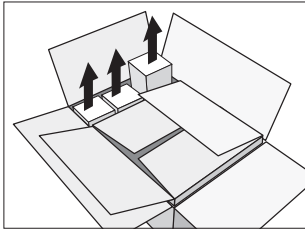


- 1 Lift out the accessories carton.

W&H packaging is environmentally friendly and can be disposed of through branch recycling companies.

However, we recommend that you keep the original packaging.

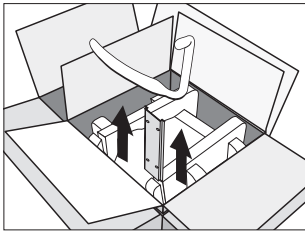

- 2 Knee control unit: Remove the holding plate and fitting for the blow out function.

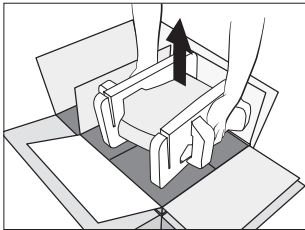

- 3 Knee control unit: Remove the holding plate and fitting for the blow out function.

3. Perfecta 300 equipment supplied (see also page 11)

Foot control unit	Knee control unit	Table control unit
<p>Control unit:</p> <ul style="list-style-type: none"><input type="radio"/> REF 05077900 230 V<input type="radio"/> REF 05172400 115 V <p><input type="radio"/> Motor cable 1,8 m REF 05114900</p> <p>optional: »E-type«</p> <ul style="list-style-type: none"><input type="radio"/> <i>E-Motor with ISO coupling LA-3E, REF 0513880</i>	<p>Control unit:</p> <ul style="list-style-type: none"><input type="radio"/> REF 05222000 230 V<input type="radio"/> REF 05222100 115 V <p><input type="radio"/> Operating controls LA-3D REF 05127000</p> <p><input type="radio"/> Holding plate REF 01170500</p> <p><input type="radio"/> Tapping screws REF 00952200</p> <p><input type="radio"/> Motor cable 1,8 m REF 05114900</p> <p>optional: »E-type«</p> <ul style="list-style-type: none"><input type="radio"/> <i>E-Motor with ISO coupling LA-3E, REF 0513880</i>	<p>Control unit:</p> <ul style="list-style-type: none"><input type="radio"/> REF 05221600 230 V<input type="radio"/> REF 05221700 115 V <p><input type="radio"/> Operating controls LA-3D REF 05127000</p> <p><input type="radio"/> Foot-operated starter L-NV REF 05038100</p> <p><input type="radio"/> Motor cable 1,8 m REF 05114900</p> <p>optional: »E-type«</p> <ul style="list-style-type: none"><input type="radio"/> <i>E-Motor with ISO coupling LA-3E, REF 0513880</i>

Perfecta 600 equipment supplied (see also page 11)

Foot control unit	Knee control unit	Table control unit
<p>Control unit:</p> <ul style="list-style-type: none"><input type="radio"/> REF 05172600 230 V<input type="radio"/> REF 05172800 115 V <ul style="list-style-type: none"><input type="radio"/> Hose for air supply REF 05250400<input type="radio"/> Motor cable 1,8 m REF 05116500	<p>Control unit:</p> <ul style="list-style-type: none"><input type="radio"/> REF 05223100 230 V<input type="radio"/> REF 05223200 115 V <ul style="list-style-type: none"><input type="radio"/> Operating controls LA-6D REF 05243200<input type="radio"/> Holding plate REF 01170500<input type="radio"/> Tapping screws REF 00952200<input type="radio"/> Fitting for blow out function REF 05148800<input type="radio"/> Hose for air supply REF 05250400<input type="radio"/> Motor cable 1,8 m REF 05116500	<p>Control unit:</p> <ul style="list-style-type: none"><input type="radio"/> REF 05222400 230 V<input type="radio"/> REF 05222500 115 V <ul style="list-style-type: none"><input type="radio"/> Operating controls LA-6D REF 05243200<input type="radio"/> Foot-operated starter with blow out function L-NV REF 05038100<input type="radio"/> Hose for air supply REF 05250400<input type="radio"/> Motor cable 1,8 m REF 05116500

Perfecta 300 and 600 equipment supplied

- Motor handpiece LA-66 REF 05201200
- Handpiece holder REF 03211500
- Chuck key REF 01125900
- Spanner REF 03202800
- W&H Service oil MD-30 REF 03304500
- Cleaning brush REF 00669400

- Mains cable REF 01343700 (Europe)

alternative:

- Mains cable REF 01343900 (USA, CAN, J) / REF 03212700 (UK, IRL) / REF 02909300 (AUS, NZ) / REF 04280600 (CH)

4. Safety notes



Please observe the following instructions under all circumstances

- > Before using the Perfecta for the first time, store it at room temperature for 24 hours.
- > Only connect the Perfecta to a socket outlet with protective earthing.
- > Never touch rotary instruments which are still rotating.
- > Never touch the chuck mechanism / twist chuck of the motor handpiece while they are still running.
- > Comply with the necessary protective measures such as the wearing of safety goggles, use of protective screens etc.
for »E-type«:
- > *Only fit the straight or contra-angle handpiece when the motor is at a complete standstill. (Perfecta 300)*
- > Always observe the contents of the safety data sheet for the material you are working with.
- > The Perfecta is intended for use at a dental technician's workstation (suction, protective equipment).



Use only suitable and serviceable tools

Use only good quality rotary tools which comply with EN ISO 1797-1. Ensure that you comply with the tool manufacturer's instructions with respect to maximum speed, maximum torque, forward and reverse movement!

Inappropriate use

Improper use, in addition to incorrect assembly, installation, modification or repairs of the Perfecta or non-compliance with our instructions invalidates all claims under warranty and any other claims.

The control unit is only permitted for use in confined spaces (indoor use)!

Safety notes



Danger zone


The control unit is not suitable for use in areas in which special conditions prevail (e.g. corrosive or explosive atmospheres).

Power failure


In the event of a power failure or if the Perfecta is switched off, the last speed set is saved and re-activated on power-up.

Intermittent operating mode S6 (4min/10min) is the designation for continuous operation with intermittent loading. The recommended loading time is 4 minutes at a running time of 10 minutes.


5. Description of foot control unit – front side

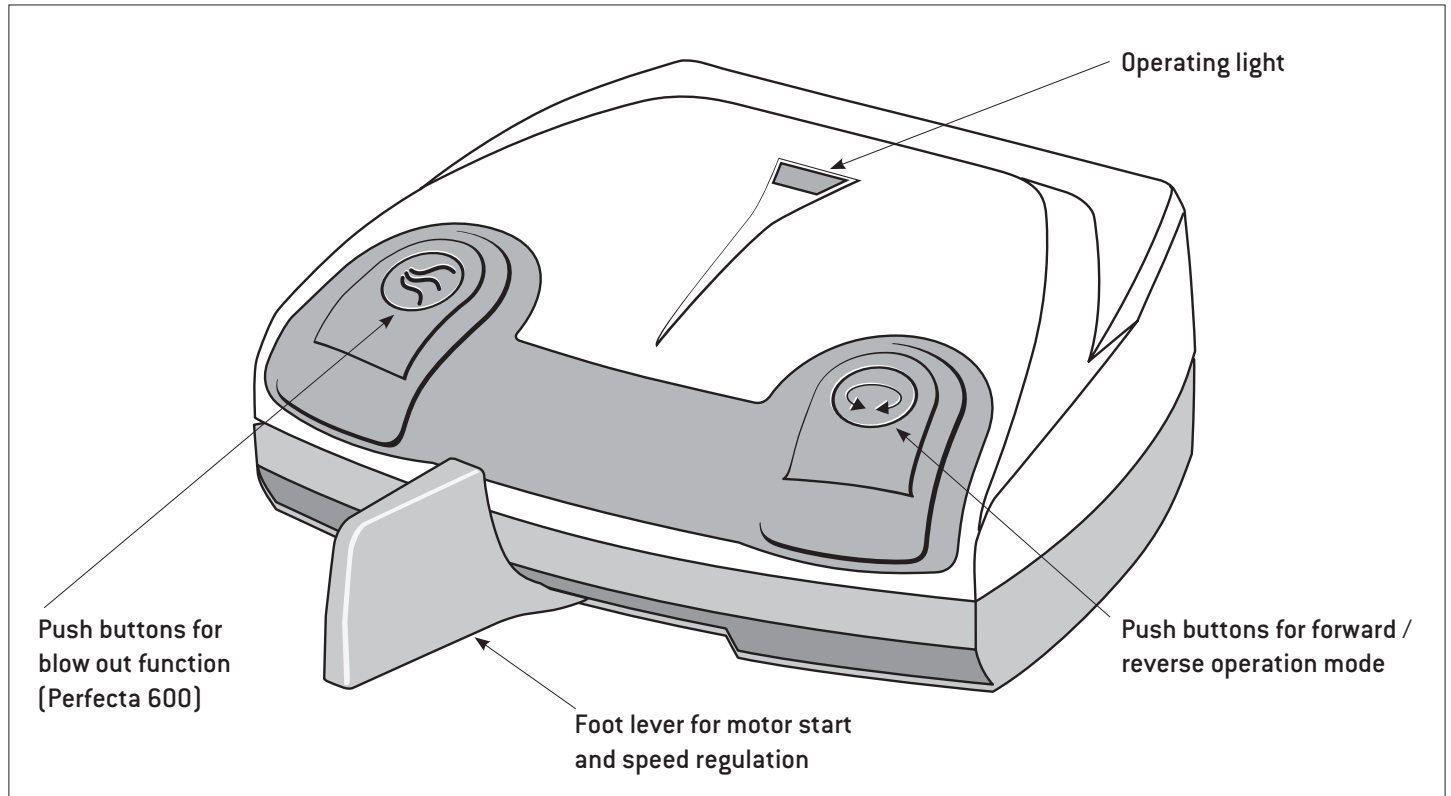


Description of foot control unit – reverse side

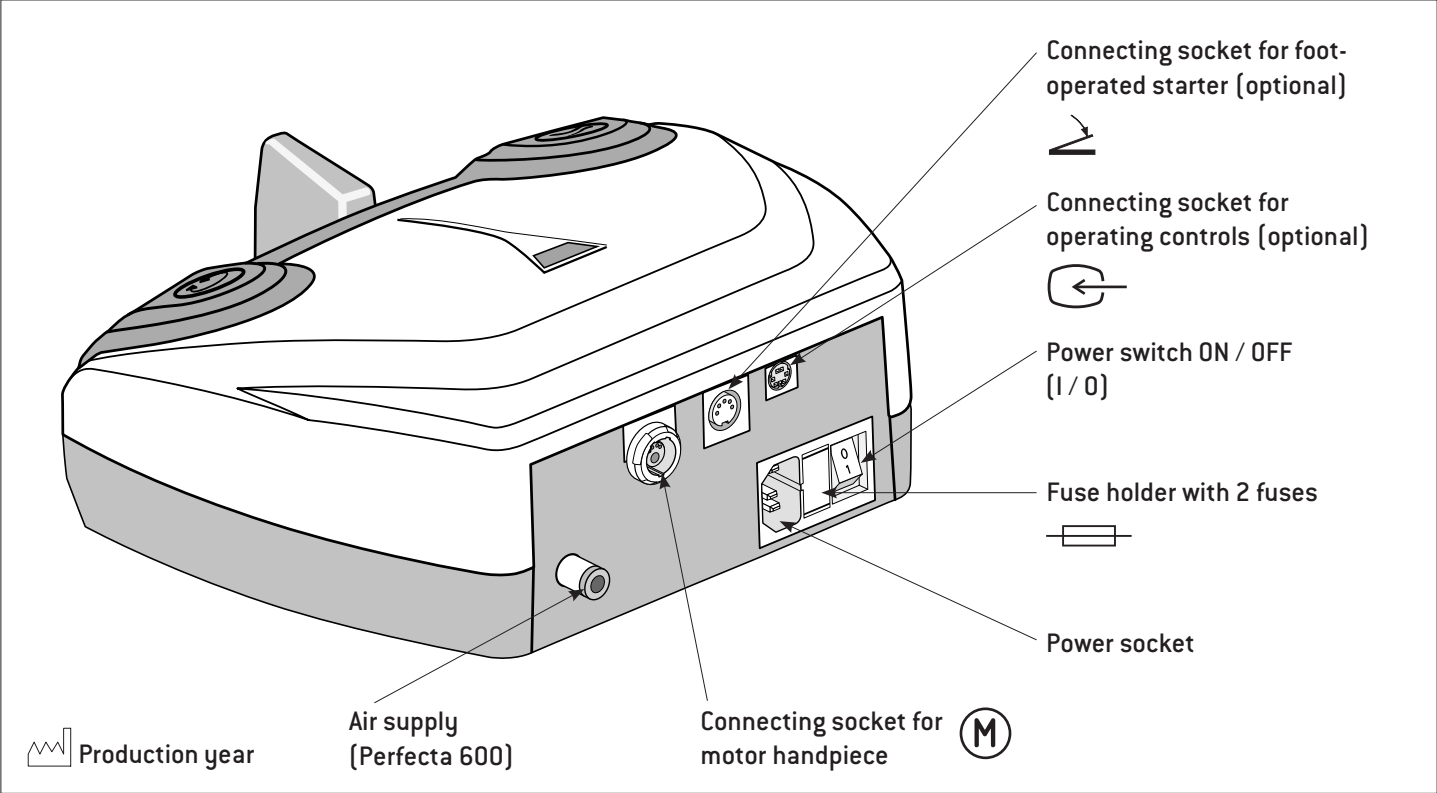


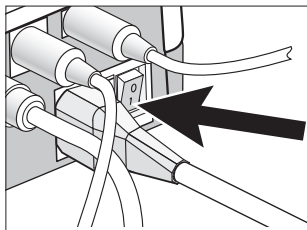
Starting operation the foot control unit

 Before connect or disconnect mains cable, motor cable, operating controls (optional), air hose (Perfecta 600), switch off the control unit.

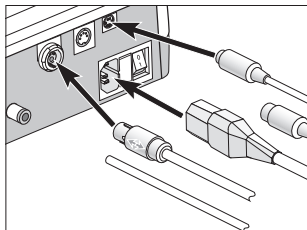

1 Connect the motor cable, operating controls (optional), air hose (Perfecta 600) and mains cable.


 Note the positioning!




2 Switch on control unit (I).


Operating the foot control unit

Reverse operation

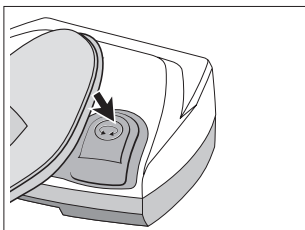


 You can switch between forward and reverse operation by pressing the button.

Changing the speed

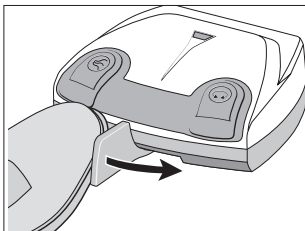


 By activating the foot lever, the values 1,000 – 40,000 rpm (1,000 – 50,000 rpm with Perfecta 600) are continuously increased / decreased.

6. Description of knee control unit

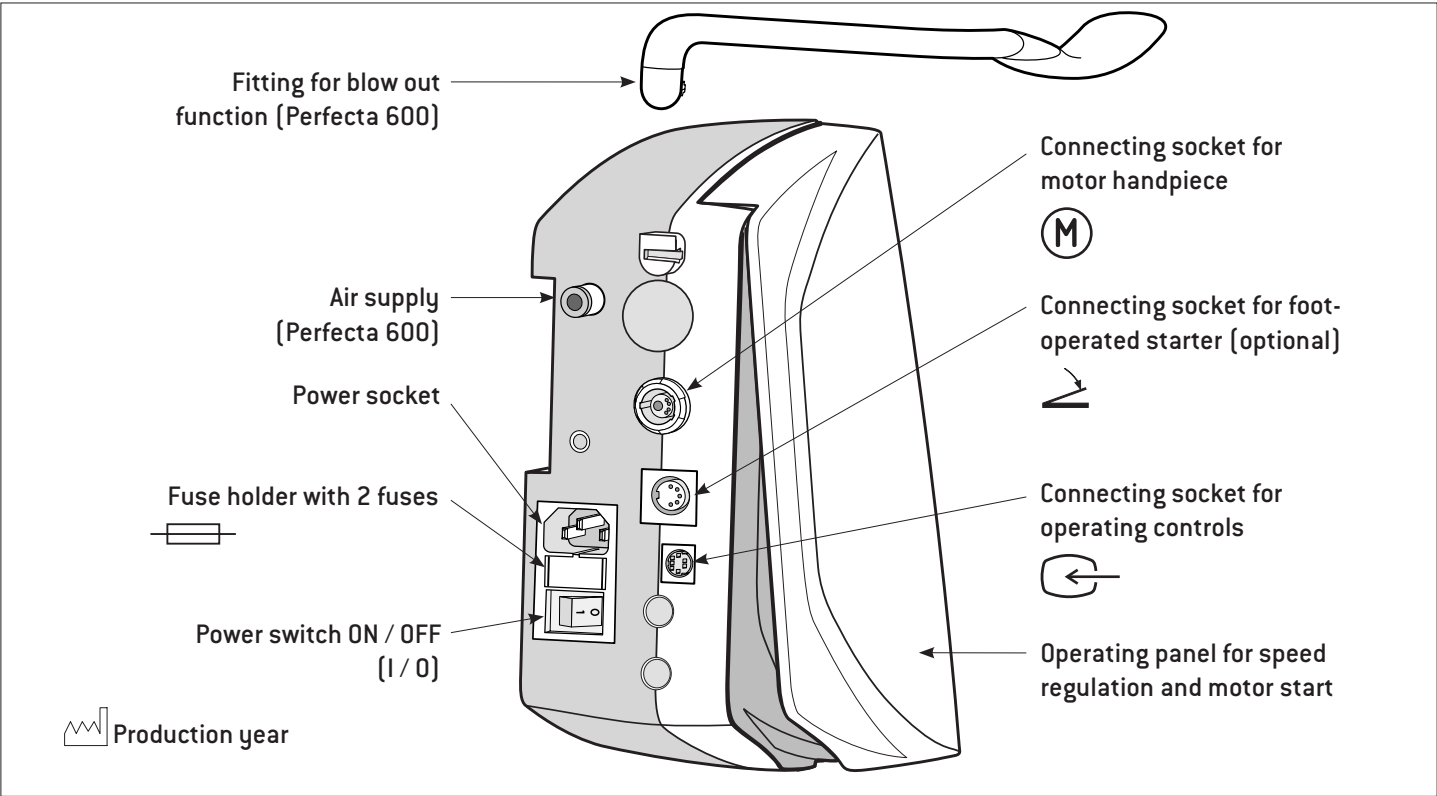

Knee control unit assembly

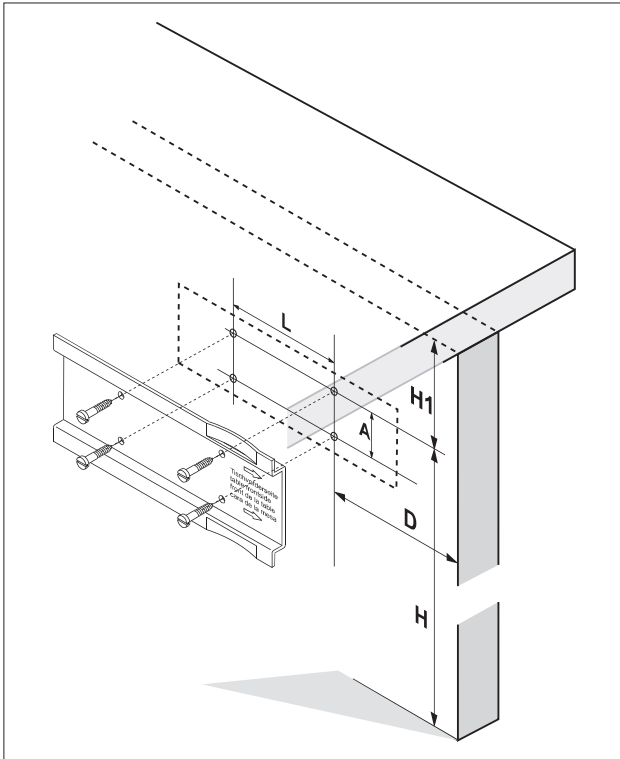

❶ Mark the screw holes with the enclosed drilling template or holding plate.

❷ Pre-drill 4 screw holes with \varnothing 3 mm.

Note dimensions:

H = 550 to 600 mm


H_1 = at least 90 mm


D = 90 mm (measured from the front edge of the bench)

L = 100 mm


A = 40 mm


Knee control unit operation


 Before connect or disconnect mains cable, motor cable, operating controls, foot-operated starter (optional), air hose (Perfecta 600), switch off the control unit.

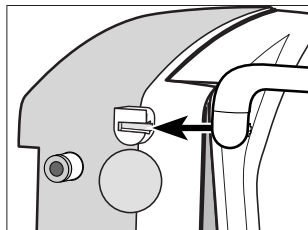


1 Push in the fitting for the blow out function up to the limit stop. (Perfecta 600).

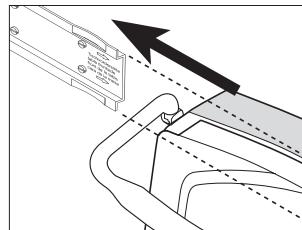


3 Insert control unit up to the limit stop on the holding plate.
 One drop of oil makes insertion easier.

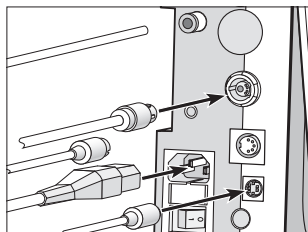


2 Connect the motor cable, operating controls, foot-operated starter (optional), air hose (Perfecta 600) and mains cable.

 Note the positioning!

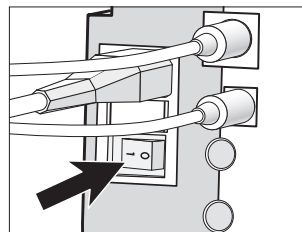



4 Switch on control unit (I).


7. Description of table control unit

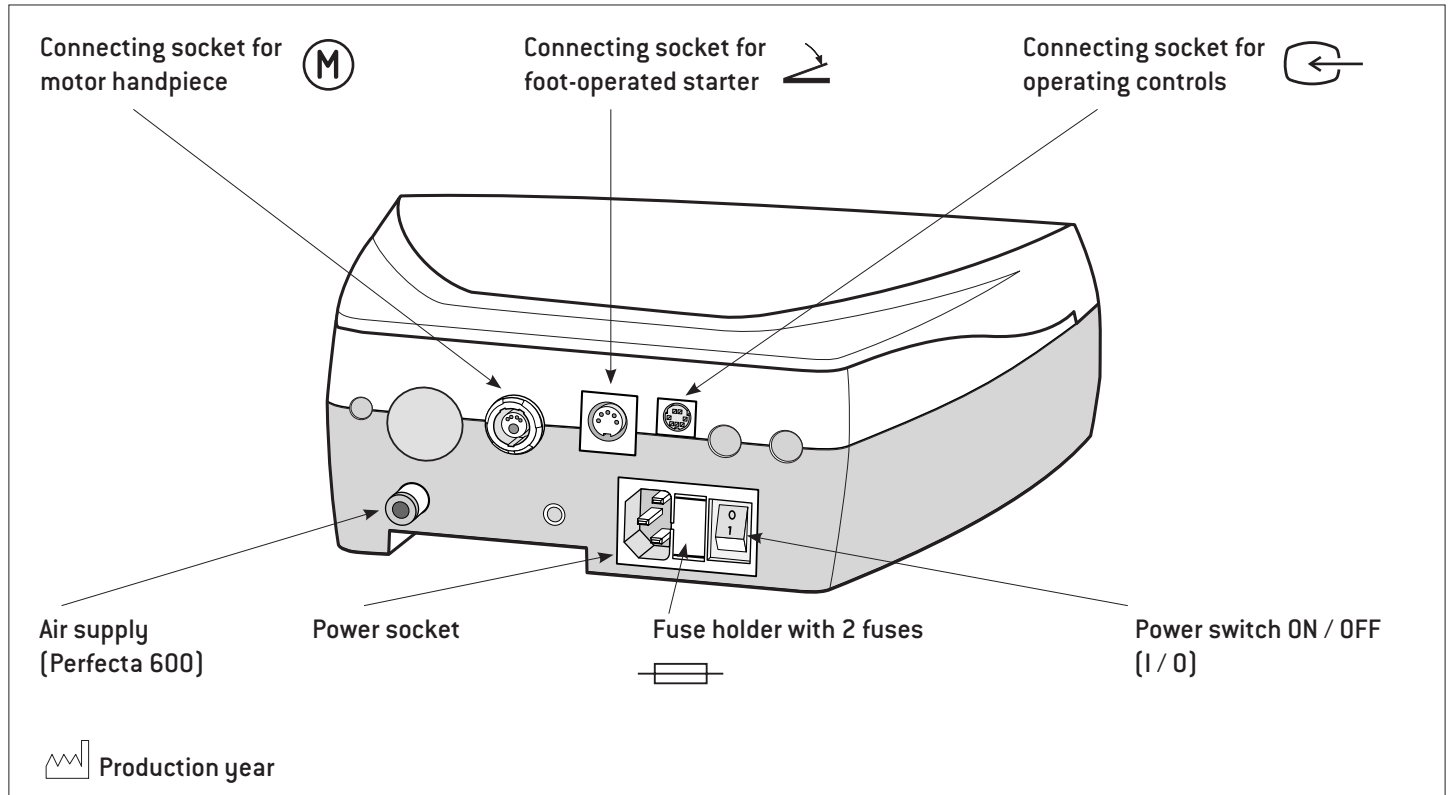


Starting operation – Table control unit

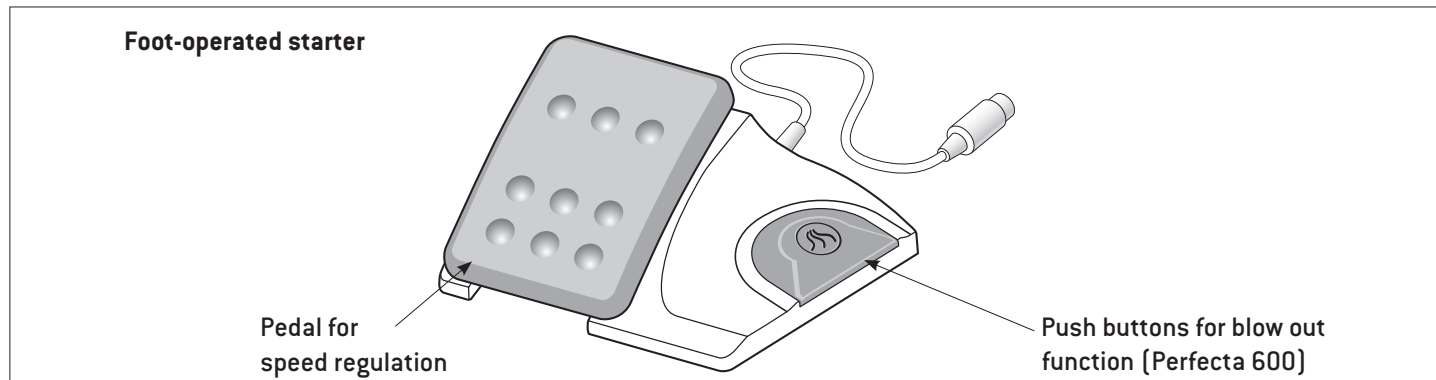

 Before connect or disconnect mains cable, motor cable, operating controls, foot-operated starter, air hose (Perfecta 600), switch off the control unit.


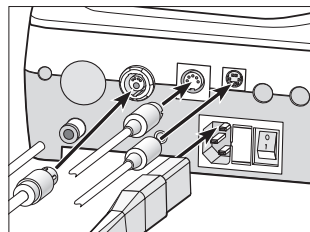


- 1** Connect the motor cable, operating controls, foot-operated starter, air hose (Perfecta 600) and mains cable. Note the positioning!

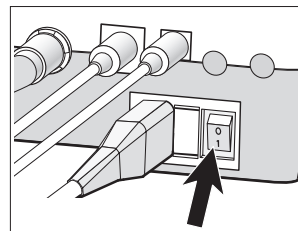

- 2** Switch on control unit (I).


8. Description of the motor handpiece LA-66 / *E-Motor »E-type«* LA-3E

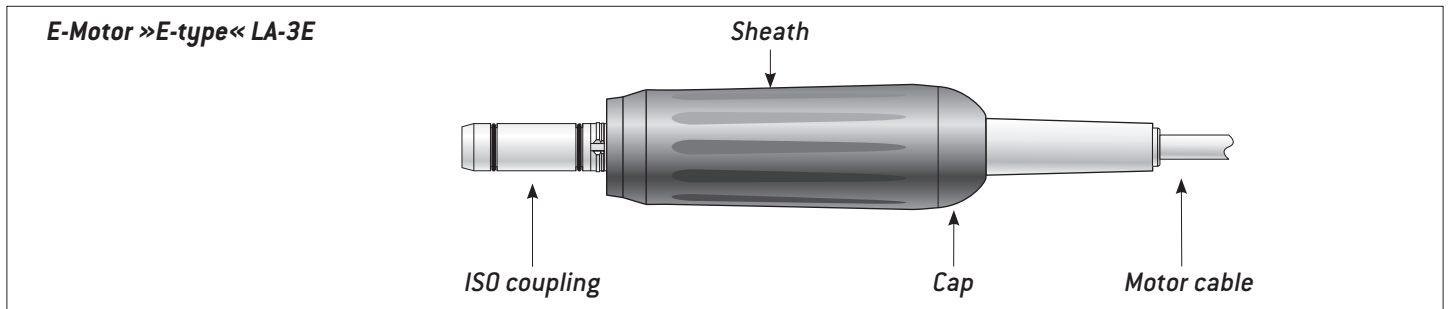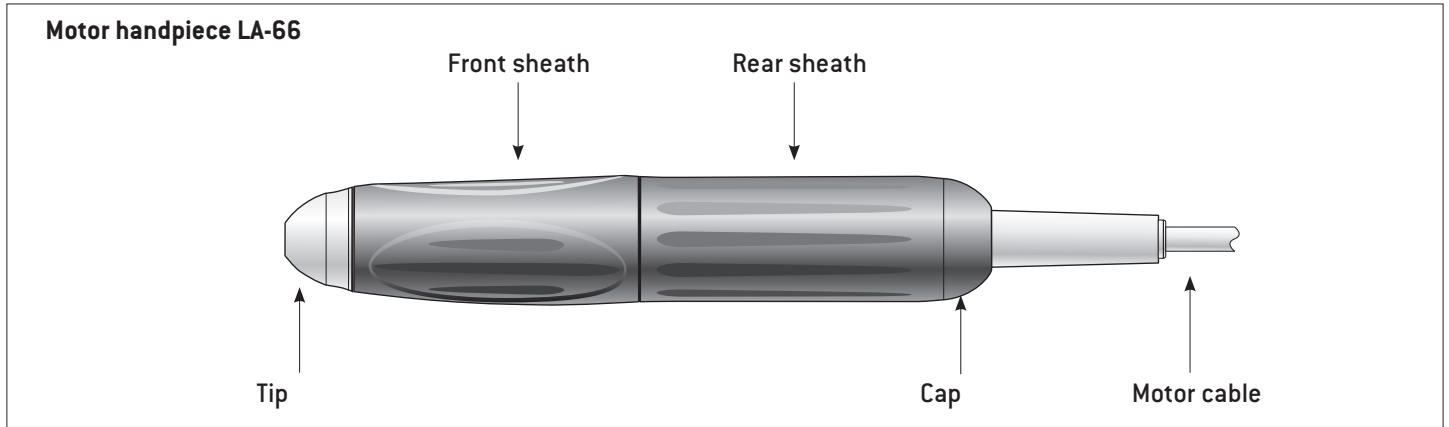


9. General operation – start motor handpiece




Start motor handpiece by pressing the foot or knee control unit or the foot-operated starter.

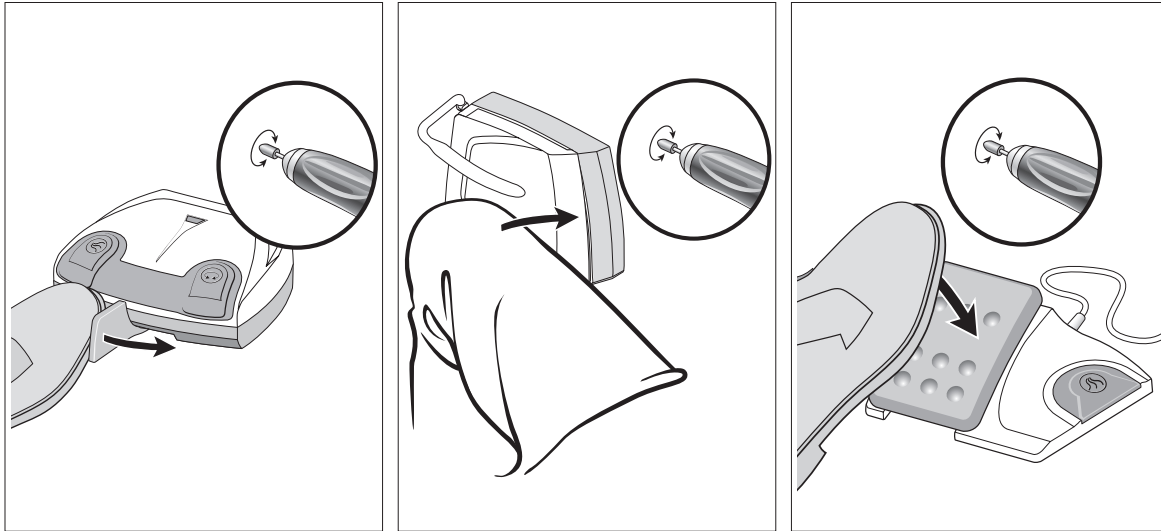


General operation – blow out function (Perfecta 600)



 Activate the blow out function by constantly pressing the button or lever.

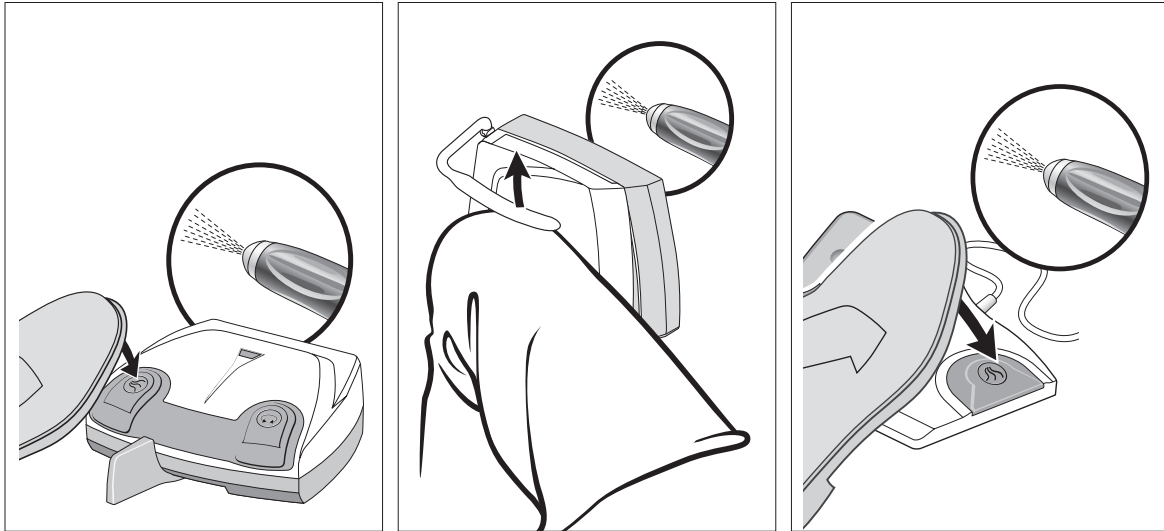


10. Description of operating controls

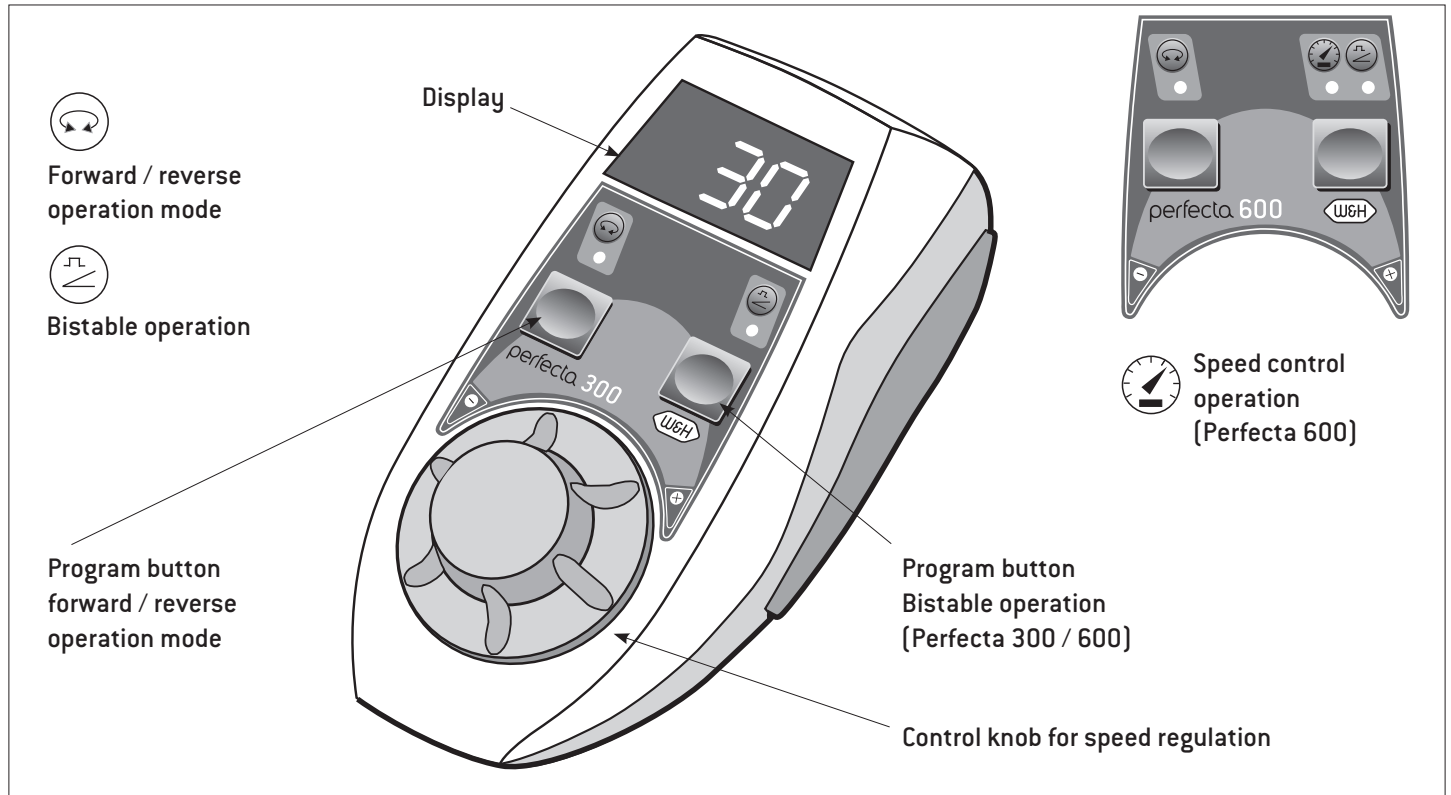

Description of the operating controls – Assemble support (optional)

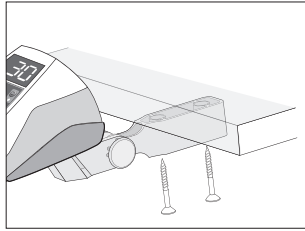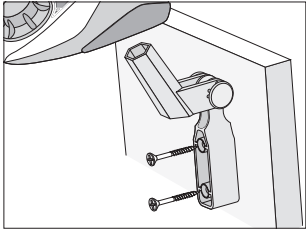


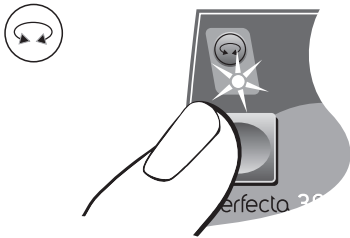
- 1 Assemble the support (optional).

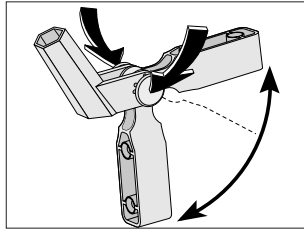


- 2 The support (optional) can be moved into a variety of assembly positions by simultaneously pressing both buttons on the joint.


11. Operating controls – Reverse operation mode

 You can switch between forward and reverse operation by pressing the program button. When switching to reverse operation, an audible signal sounds and the LED illuminates.




Operating controls – Changing the speed


 By turning the PLUS / MINUS control knob, the values 1,000 – 40,000 rpm are continuously increased / decreased.

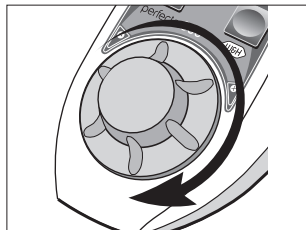


1 Increase speed.

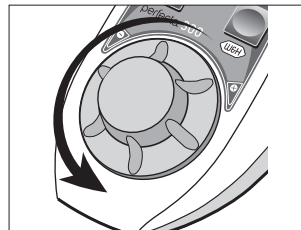


2 Decrease speed.


 Set the speeds from 40,000 to 50,000 rpm. (Perfecta 600)

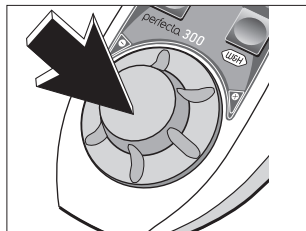


1 Press the control knob.

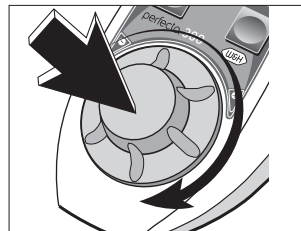

2 Keep the control knob pressed down and turn it.

 Pre-selected maximum speed must not be exceeded during motor start.

Operating controls – Bistable operation


 During bistable operation, the selected maximum speed is automatically reached during motor start. The motor handpiece runs independently.




Deactivation of bistable operation: Press program button x 1 (Perfecta 600: Press the button x 2).



- 1 Press the button.
The LED illuminates.

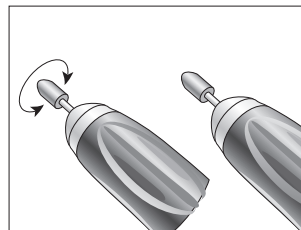

- 3 Start or stop the motor handpiece by tapping the pedal, lever or flap.

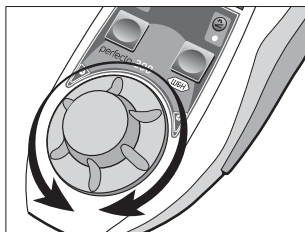

- 2 Set the speed.


Operating controls – Speed control operation (Perfecta 600)




During speed control operation, the controlled speed is saved and automatically maintained.

The motor handpiece runs independently.


Deactivation of speed control operation: Press the program button x 1.

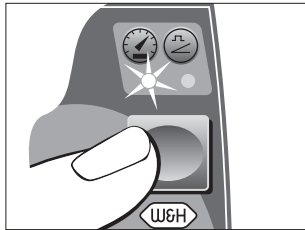

- 1 Press the button x 2.
LED illuminates.

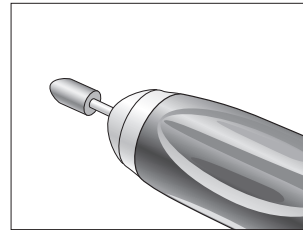


- 3 Stop the motor handpiece by tapping the pedal, lever or flap.

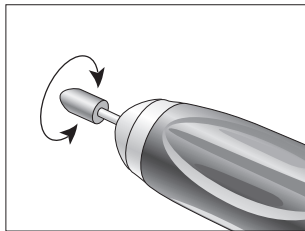


- 2 Start the motor handpiece until the required speed is reached. After approx. 2 seconds, an audible signal sounds and the required speed is saved.


12. Removing and assembling the motor handpiece

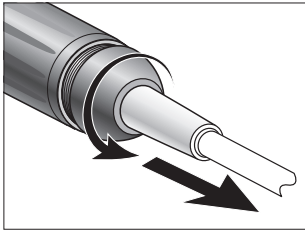


❶ Unscrew the cap.

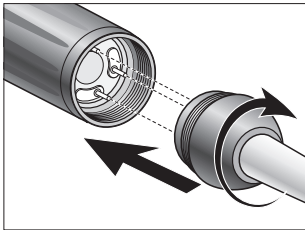


❷ Assemble and tighten the cap.

 Note the positioning!


13. Change the rotary tool

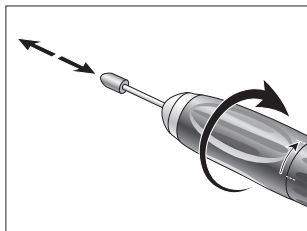


- 1 Turn the front sheath clockwise up to the limit stop. Push in the rotary tool up to the limit stop or remove.

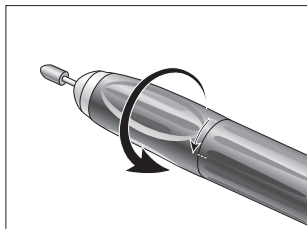

- 2 Turn the front sheath anti-clockwise until it engages audibly.

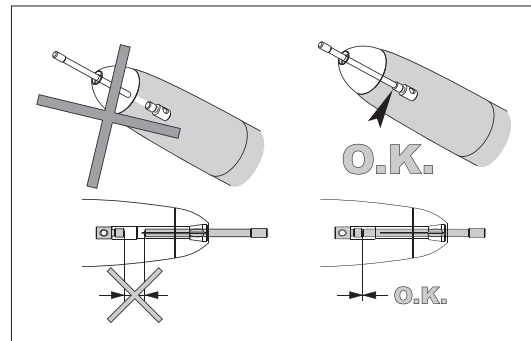

 When the chuck is open, the motor handpiece is blocked. In the event that the motor handpiece starts accidentally, the electronic system switches off.

Test run


- > Start the motor handpiece.
- > If you observe problems (e.g. vibrations, unusual noise, overheating) contact your authorized W&H service partner (see page 45).


14. Cleaning




The cleaning of the Perfecta (control unit), motor handpiece, operating controls (optional), foot-operated starter (optional) can take place by means of a dry cloth.

15. Cleaning / changing the chuck


 Chuck key, spanner, cleaning brush are located on the underside of the handpiece holder.

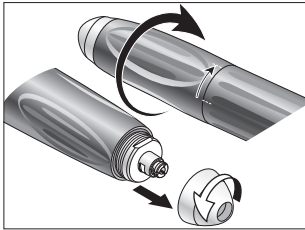

- 1 Turn the front sheath clockwise up to the limit stop.
Unscrew the tip anti-clockwise.

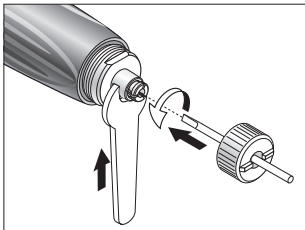


- 2 Insert the chuck key up to the limit stop in the chuck and unscrew anticlockwise.
 Hold the shaft firmly with the spanner.

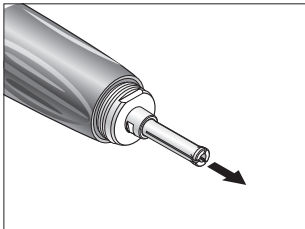


- 3 Remove the chuck.


Cleaning / changing the chuck

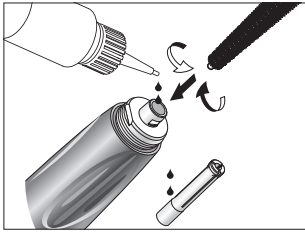

- 1 Clean the inside and outside of the shaft using the cleaning brush.
Apply 2 drops of oil into the hole in the shaft and outside of the chuck.

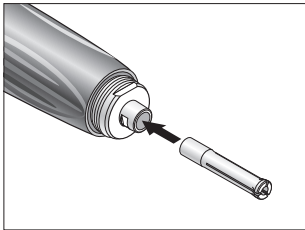

- 2 Insert the chuck.

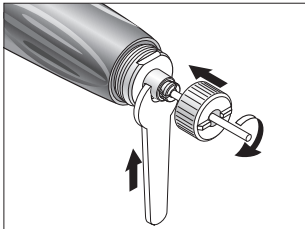

- 3 Insert the chuck key up to the limit stop in the chuck and screw clockwise.
Hold the shaft firmly with the spanner.

16. Error messages (as shown on display)

Error No	Description	Remedy
E00	Electronics overheating – safety shutdown	Switch off the equipment, allow equipment to cool for at least 10 minutes and then re-start
E01	Motor handpiece overloading – Drive blocked	Connect motor handpiece or close the chuck mechanism
E07	Foot-operated starter error initialising	Switch off the equipment, re-start, do not activate the foot / knee control when switching on
E09	Foot-operated starter error Speed regulator (foot / knee control unit) error	Switch off the equipment, check the connection of the foot control, restart
E19	Run limiting control	Switch off the equipment and re-start
E99	System failure	Switch off the equipment and re-start



If one of the error messages described above cannot be rectified by switching off the Perfecta and then switching it on again, the equipment must be checked by an authorized W&H service partner (see Page 45). If a total failure of the equipment occurs caused by external circumstances, the equipment must be switched off and then on again.

17. W&H Accessories

Use only original W&H accessories / spare parts
or accessories approved by W&H

00669400	Cleaning brush
01125900	Chuck key \varnothing 2,35 mm
01126000	Chuck key \varnothing 3 mm
03202800	Spanner
03205500	Chuck \varnothing 2,35mm
03205600	Chuck \varnothing 3 mm
03211500	Handpiece holder
03304500	W&H Service oil MD-30
04014700	Fuse T1,25L
01199100	Fuse T2A
05114900	Motor cable 1,8 m (Perfecta 300)
05116500	Motor cable 1,8 m (Perfecta 600)
05138000	Support for operating controls (optional)
05201200	Motor handpiece LA-66
05138800	<i>E-Motor with ISO coupling LA-3E</i>
05250400	Hose for air supply
05038100	Foot-operated starter with / without blow-out function L-NV
05127000	Operating controls LA-3D
05243200	Operating controls LA-6D

18. Servicing

Repairs

If a defect occurs, always return all the equipment, due to the fact that with motor malfunctions, an inspection of the electronic controls is also necessary!

We recommend that only an authorized W&H service partner should undertake this servicing and checking.

Returns

- > Refer all questions to an authorized W&H service partner (see Page 45).
- > Always return equipment in the original packaging!
- > Do not coil the cable around the motor handpiece and do not twist or kink the motor cable! (Risk of damage)

Regular checking of Perfecta and accessories

Regular servicing of function and safety including the accessories is necessary and should be carried out at least once every three years, unless shorter intervals are prescribed by law.

We recommend that only an authorized W&H service partner (see page 45) should undertake this servicing and checking.

19. Technical Data

	Perfecta 300	Perfecta 600
Mech. output power	95 W	160 W
Torque	5,5 Ncm	7,8 Ncm
Speed	1.000 – 40.000 Upm	1.000 – 50.000 Upm
Power input	160 W	200 W
Supply voltage	100 – 130 VAC / 220 – 240 VAC	
Rated current	0,2 – 1,6 A / 0,1 – 0,8 A	
Supply fuse	250 V – T2A / 250 V – T1,25L	
Voltage tolerance	+/- 10 %	
Frequency	50 – 60 Hz	
Operating mode	S6 (4/10min) Continuous operation with intermittent loading	
Noise	< 55 dBA	
Vibration	< 2,5 m/s ²	
Height / width / depth		
foot control unit	without lever: 97 / 253 / 202 mm, with lever: 97 / 253 / 275 mm	
knee control unit	without fitting: 236 / 108 / 282 mm, with fitting: 236 / 187 / 304 mm	
table control uni	88 / 236 / 282 mm	
Weight foot / knee / table control unit	2,5 / 3,7 / 4,0 kg	3,5 / 4,7 / 5,0 kg
Blown out air pressure (input)		max. 6 bar (87 psi / 600 kPa)
Chuck diameter	2,35 / 3,0 mm	
Bur concentricity accuracy	≤ 0,02 mm	

Technical Data

Physical characteristics

Temperature in storage:	-40 °C [-40 °F] to +70 °C [+158 °F]
Air humidity in storage:	8 % to 80 % [relative], non-condensing
Temperature in operation:	+5 °C to +40 °C
Air humidity in operation:	maximum 80 % [relative] with a temperature of up to +31°C, decreasing arithmetically up to a maximum of 50 % [relative] with a temperature of up to +40°C
Pollution degree:	2
Overvoltage degree:	II
Altitude:	up to 2,000 m above sea level

20. Recycling and Disposal

Recycling

W&H considers that it has a special duty towards the environment. The Perfecta equipment along with its packaging has been designed to be as environmentally friendly as possible.



Disposal of the Perfecta (control unit), operating controls, foot-operated starter, motor handpiece

Follow your country-specific laws, directives, standards and guidelines for the disposal of used electrical devices.

Disposal of the packaging material

All packaging materials have been selected according to environmentally compatible and disposal aspects and can be recycled. Please send old packaging materials to the relevant collection and reprocessing system. In this way, you will contribute to the recycling of raw materials and the avoidance of waste.

Letter of indemnity

This W&H product has been manufactured with great care by highly qualified specialists. A wide variety of tests and controls guarantee faultless operation. Please note that claims under warranty can only be validated when all the directions in the Instructions for use have been followed.

As manufacturer, W&H is liable for material or manufacturing defects within a warranty period of 24 months from the date of purchase.

We accept no responsibility for damage caused by incorrect handling or by repairs carried out by third parties not authorized to do so by W&H!

Claims under warranty – accompanied by proof of purchase – must be sent to the vendor or to an authorized W&H service partner. The provision of service under warranty extends neither the warranty period nor any other guarantee period.

24 months warranty

CE-Declaration of conformity

W&H hereby declares that the product complies with the following regulations:

Machinery – Regulation (EU) 2023/1230

Electromagnetic Compatibility – Directive 2014/30/EU

ROHS – Directive 2011/65/EU

The full text of the EU declaration of conformity is available at the following internet address <https://wh.com>.

Authorized W&H service partners

Find your nearest authorized W&H service partner at <https://wh.com>
Simply go to the menu option »Service« for full details.

Or simply scan the QR code.





W&H Dentalwerk Bürmoos GmbH
Ignaz-Glaser-Straße 53, 5111 Bürmoos, **Austria**

t +43 6274 6236-0,
office@wh.com

f +43 6274 6236-55
wh.com

Form-Nr. 50588 AEN
Rev. 011 / 07.10.2024
Subject to alterations