

#whdentalwerk



video.wh.com

Now at
your dealer
or wh.com



Efficient. Flexible. Gentle.

Piezo Scaler
handpieces



Piezo Scaler

with first-class performance

The W&H ultrasonic scaler stands out for its fast removal of calculus and gentle, minimally invasive application. It offers optimal comfort and is flexible in its use thanks to the efficient W&H tip system. State-of-the-art piezo technology improves efficiency and tactile feedback in all clinical indications.



W&H tip system

The tip system, developed with end users, is perfectly matched to the requirements of the respective application areas and the individual treatment steps.



Maximum efficiency

High-performance and gentle at the same time for a minimally invasive application and painless treatment.

Easy to clean

Both Piezo Scaler handpieces can be easily dismantled for simple and improved cleaning and both are thermo washer disinfectable and sterilizable.




Handy and comfortable

The shape of the handpiece and its light weight ensure maximum comfort.



5x ring LED

Daylight-quality illumination of the treatment site thanks to 5 LEDs with the PB-5 L or PB-5 L S handpiece.

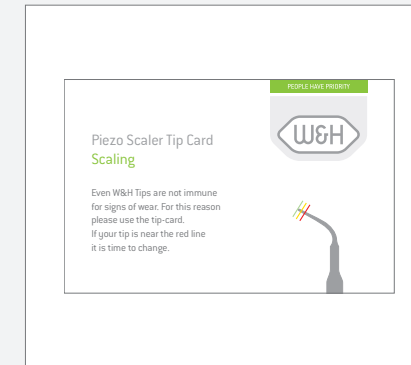

Useful accessories

Change tips simply and safely with the tip changer and check them using the tip card.



Treatment site always cooled

The integrated spray ensures constant and effective cooling of the treatment site.

Compact, high-quality tip system for Piezo scalers

Universal and special tips developed with end users and tailored to the clinical requirements of each area of application, the individual treatment stages, and the technical requirements of W&H Piezo technology.



Universal tips

- 1U, 2U, 3U, 4U
- 1US, 2US

For supragingival scaling in all quadrants.



Implant Clean tip

- 1I

For gentle cleaning of implants and the associated restorations.




Periodontal tips

- 1P, 3Pr/3PI, 4P, 5Pr/5PI

For initial and supportive periodontitis treatment.

Versatile application

Thanks to infinitely variable power control, the W&H Piezo scalers can be easily adjusted to each application and tip. From prophylaxis and periodontology, implant maintenance or endodontics, or even the micro-preparation of dental restorations, the potential applications are particularly diverse.

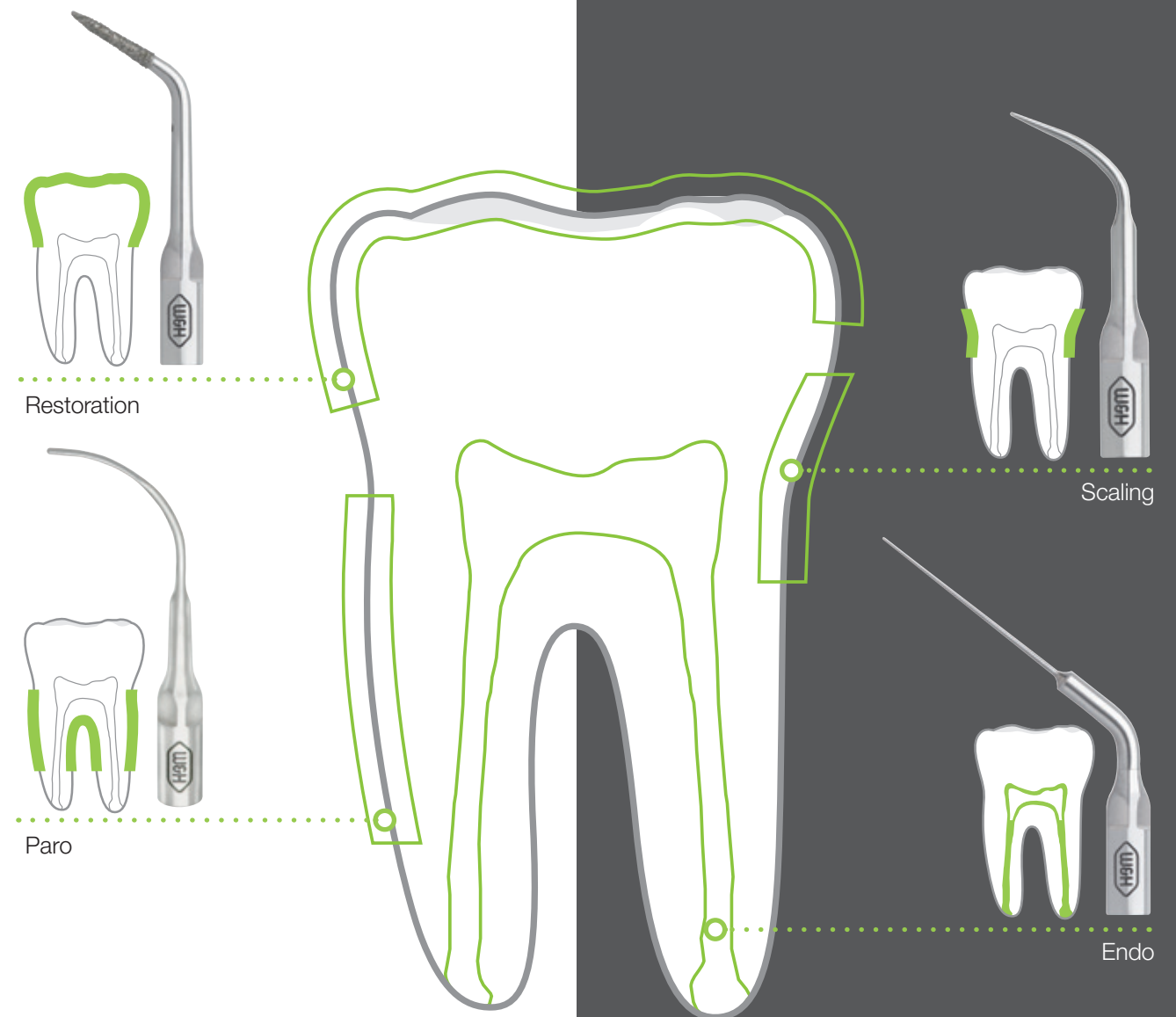

©Krapf-Romana

W&H Piezo Scalers

The W&H Piezo Scalers can be optimally integrated into your personalised workflow. Choose a Proxeo Ultra Piezo Scaler or take advantage of a Built-in Solution – convenient integration in your dental unit.



PB-530

The W&H Piezo Scaler with wireless foot control for increased freedom of movement and the ultimate in user comfort.



PB-520 / PB-510

The compact models of the Proxeo Ultra Piezo Scalers are certain to be a hit with or without a tank.



LED handpiece

The ring LED allows daylight-quality illumination of the treatment site.



Built-in Set analogue

Get your treatment unit ready for the future. With the W&H Built-in Solutions, such as the PB-5 L Built-in Set analogue. The new-generation Piezo Scaler Built-in Solution – compatible and combinable.



Tip-top selection

The Proxeo Ultra is available with two different thread types.

Technical data

Piezo Scaler Handstück	PB-3 / PB-3 S	PB-5 L / PB-5 L S
Light:	without light	LED ring
Weight:	52.5 g	55 g
Length without tip:	104.3 mm	135 mm (with coupling)
Diameter:	20.2 mm	18.5 mm

Built-in Set analog	PB-3 / PB-3 S	PB-5 L / PB-5 L S
Piezo Controller:		PC-1.0
Electrical connection set:		✓
Control knob:		✓
Supply hose:	VP-3 (1.8 m)	VP-5 (1.8 m)
Handpiece:	PB-3	PB-5 L
Tip changer with tip inserted:	TW-1: 1U, 1P	TW-1: 1U, 2U, 1P

Tania Eberle

dental hygienist, Germany

“The W&H Piezo Scaler makes light work of what is often not possible by hand. A very convincing product!”



Stand-alone solutions

Detailed information on the W&H Piezo Scaler stand-alone solutions Proxeo Ultra can be found at wh.com

Compatibility of tips with W&H handpieces

	W&H/EMS/Mectron		W&H/NSK/Satelec*	
	PB-5 L	PB-3	PB-5 L S	PB-3 S
Proxeo Ultra	X		X	
Built-in	X	X	X	X

* W&H tips with Satelec thread (e.g. 1US, 2US) are compatible with the W&H handpieces PB-5 L S, PB-3 LED S and PB-3 S.

Note: EMS®, Mectron®, NSK® and Satelec® are trademarks of third parties not affiliated with W&H Dentalwerk Bürmoos GmbH. When using third-party tips, the user has to adjust the performance parameters of the W&H Piezo unit accordingly.

Manufacturer:

W&H Dentalwerk Bürmoos GmbH
 Ignaz-Glaser-Straße 53, Postfach 1
 5111 Bürmoos, **Austria**
t +43 6274 6236-0
f +43 6274 6236-55
 office@wh.com
wh.com